



└─ #500-094, ICS LOWER BELT CLIP CUSHION FRAME ICS ROD



ILL #3



- #500-094, ICS LOWER BELT BUCKLE - GROMMET






#600-640, CUSHION KEY

#600-640, CUSHION KEY



-GROMMET FILLER PART GROMMET ASSY



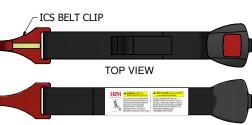

BOTTOM VIEW #500-094, ICS LOWER BELT BUCKLE (w/ REQUIRED LABEL)

RECALL #18E-083 SERVICE REPAIR PROCEDURE #1801-0012

PARTS/ITEMS NEEDED:

- 1. 500-094 ICS LOWER BELT BUCKLE w/ LABEL
- 2. CUSHION KEY, #600-640
- 3. FLAT HEAD SCREWDRIVER
- 4. WEDGE OR CLOTH, (INSTALLER PREFERENCE)

5. VISE GRIPS OR CLAMP w/ RUBBER ENDS, (INSTALLER PREFERENCE)

STEPS TO REPLACE BELT:

1. FIRST CHECK WHETHER ICS LOWER BELT BUCKLE NEEDS REPLACED. IF CURRENT ICS BELT DOES NOT HAVE LABEL, PROCEED WITH REPLACING BELT.

2. ASSURE ALL NEEDED PARTS/ITEMS ARE AVAILABLE.

3. UNLOCK CUSHION AND PLACE IN UPWARD POSITION. TO UNLOCK PLACE KEY IN PLUNGER, TWIST TO SET AND PULL OUT TO DISENGAGE. SEE ILL #5

NOTE: CAN KEEP CUSHION UNLOCKED TO ALLOW FOR QUICKER REPLACMENT, (INSTALLER PREFERENCE).

4. TO ACCESS #500-094 ICS LOWER BELT BUCKLE ATTACHMENT POINT, OPEN BOTTOM OF CUSHION COVER BY UNFASTENING HOOK AND ARROW ON SIDE OF CUSHION WHERE ICS BELT IS LOCATED. SEE ILL #2.

NOTE: LOOSEN SIDE ONLY WHERE ICS BELT IS TO AVOID HAVING TO CLOSE ENTIRE CUSHION COVER. USE VISE GRIPS/CLAMP W/ RUBBER ENDS TO KEEP CUSHION COVER FROM FULLY OPENING. WHEN (2) ICS POSITIONS, INSTALL ONE AT A TIME. SEE ILL #6 (INSTALLER PREFERENCE)

5. LOCATE ICS BELT CLIP AND UNHOOK. WHEN (2) ICS POSITIONS, INSTALL ONE AT A TIME. SEE ILL #2. 6. ROTATE CUSHION INTO DOWNWARD POSITION.

7. PULL BUCKLE END OF BELT, THE PULLING MOTION MAY LOOSEN GROMMET FILLER PART AND ALLOW ENTIRE BELT TO BE REMOVED.

NOTE: IF GROMMET FILLER PART DOES NOT LOOSEN, A FLAT HEAD SCREWDRIVER CAN BE USED IN AID TO REMOVE GROMMET FILLER PART.

8. LOCATE REPLACEMENT ICS BELT 500-094 w/ LABEL.

9. GUIDE ICS BELT CLIP FIRST THROUGH GROMMET, ASSURE BELT ORIENTATION IS CORRECT.

10. ROTATE CUSHION IN UPWARD POSITION.

11. ATTACH ICS BELT CLIP INTO CUSHION FRAME ICS ROD, ASSURE BELT AND BELT CLIP ARE ON BOTTOM SIDE OF CUSHION FOAM AND FLEXMAT.

12. TO CLOSE CUSHION COVER LOCK HOOK AND ARROW. WHEN (2) ICS POSITIONS, CLOSE COVER AND REPEAT ON OPPOSITE SIDE STARTING WITH STEP #1.

13. ROTATE CUSHION INTO DOWNWARD POSITION.

14. PLACE GROMMET FILLER PART BACK INTO GROMMET, ASSURE FILLER PART IS FLUSH WITH REMAINING GROMMET ASSY.

15. ASSURE CUSHION IS LOCKED IN THE DOWNWARD POSITION.

