

Part 573 Safety Recall Report

19V-271

Manufacturer Name : Mobility Specialists, Inc.**Submission Date :** APR 03, 2019**NHTSA Recall No. :** 19V-271**Manufacturer Recall No. :** 18E-044**Manufacturer Information :**

Manufacturer Name : Mobility Specialists, Inc.

Address : 490 Capricorn Street

Brea CA 92821

Company phone : 877-777-5438

Population :

Number of potentially involved : 4

Estimated percentage with defect : 100 %

Vehicle Information :

Vehicle 1 : 2014-2017 Ford E150 , Transit

Vehicle Type : BUSES, MEDIUM & HEAVY VEHICLES

Body Style : VAN

Power Train : GAS

Descriptive Information : is recall applies to approximately 4,262 Ricon S Series and K Series Titanium Wheelchair Lifts, part numbers K200, K201, K550, K551, S200, S201, S550, S551 no dashes. The recall population includes all units of these models beginning January 1, 2012, until May 9, 2018.

Production Dates : DEC 01, 2014 - AUG 29, 2017**VIN Range 1 :** Begin : 1FTNS1EW8EDB18574 **End :** WD3PE7DC3FP139403 Not sequential**Description of Defect :**

Description of the Defect : If the position input cam fails and the lift operator continues to press the “up” button on the operating pendant while the lift is occupied, the cutoff switch is dis-abled and the platform can travel past the vehicle floor height, potentially causing the occupant to tip inwards, toward the vehicle.

FMVSS 1 : NR

FMVSS 2 : NR

Description of the Safety Risk : If the position input cam fails and the lift operator continues to press the “up” button on the operating pendant while the lift is occupied, the cutoff switch is dis-abled and the platform can travel past the vehicle floor height, potentially causing the occupant to tip inwards, toward the vehicle.

Description of the Cause : NR

Identification of Any Warning that can Occur : NR

Supplier Identification :

Component Manufacturer

Name : RICON CORPORATION
Address : 1135 AVIATION WAY
SAN FERNANDO CALIFORNIA 91340
Country : United States

Chronology :

If the position input cam fails and the lift operator continues to press the “up” button on the operating pendant while the lift is occupied, the cutoff switch is dis-abled and the platform can travel past the vehicle floor height, potentially causing the occupant to tip inwards, toward the vehicle.

Description of Remedy :

Description of Remedy Program : Ricon will provide instructions and materials for replacing defective parts. This remediation is to be conducted by Ricon Authorized dealers or Ricon personnel only.

How Remedy Component Differs from Recalled Component : NR

Identify How/When Recall Condition was Corrected in Production : NR

Recall Schedule :

Description of Recall Schedule : RICON HAS SENT OUT NOTICES TO ALL OUR CUSTOMERS WITH DEFECTIVE LIFTS
Planned Dealer Notification Date : APR 03, 2019 - NR
Planned Owner Notification Date : APR 03, 2019 - NR

* NR - Not Reported